



EASYCAP

EEG Recording Caps and Related Products

EASYCAP GmbH

Am Anger 5

82237 Würthsee

GERMANY

Tel +49 8153 88702-00


www.easycap.de

info@easycap.de

160Ch Standard Cap for actiCAP for actiCHamp

- Green Holders: Label 1-32, Hardware Channel 1 – 32
- Yellow Holders: Label 1-32, Hardware Channel 33 - 64
- Red Holders: Label 1-32, Hardware Channel 65 - 96
- White Holders: Label 1-32, Hardware Channel 97 - 128
- Blue Holders: Label 1 - 32, Hardware Channel 129 - 160
- Black holder: Label & hard-wired Gnd

Electrode Names and Number Labels

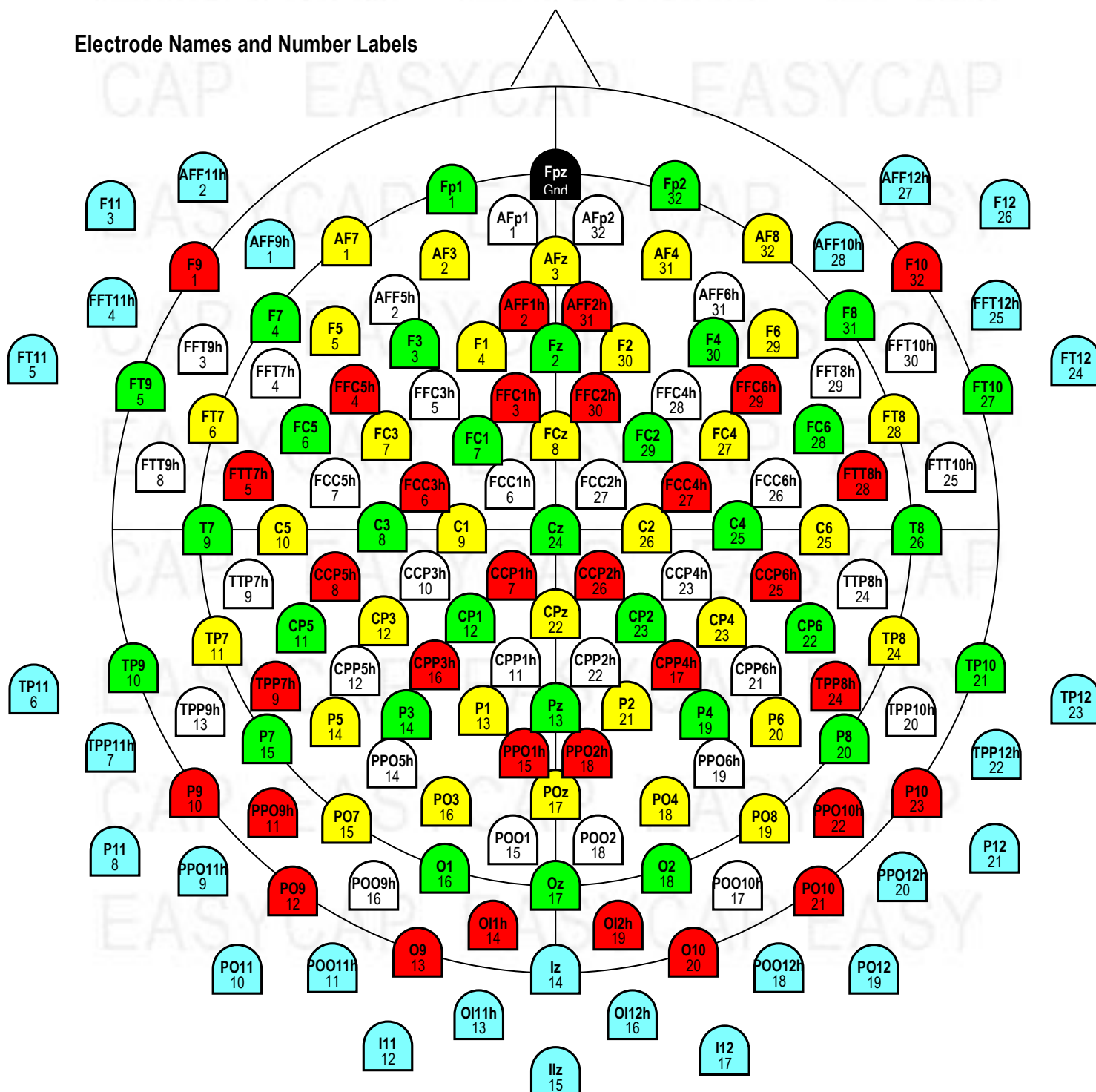

Details

For ordering please give **Article Number, Cap Cut, Cap Fabric, and Size**
(Example: CAP-160, C-Cut, High Precision, 58):

- Article Number: **CAP-160**
- Cap Cut: **C-Cut** or **A-Cut**
- Cap Fabric: **High Precision** or **High Comfort**
- Size (given in cm head circumference):
 - Adult caps: **54, 56, 58, 60, 62, 64** (average male: 58, average female: 56)
 - Children caps: **52** (5-10 years), **54** (11-14 years)

The catalogue-number comprises the cap as described. For further information about accessories or consumables, please visit our website or contact our local distributor.

Cap

White Infracerebral Cap with integrated chin belt.



This cap type is fixed by
the integrated chin belt,

no chest belt is
necessary



If the chin belt presses
uncomfortably against the
larynx,

it can be further supported
by the additional frontal
chin belt

The holders are mounted according to the figure (1-160 transparent, Gnd black). The positions are chosen from 10%-System and from 5%-System¹. Positions TP9/TP10 approximately meet left and right mastoids.

Each cap comes with 4 loose holders (blue 29 – 32) for use with double-sided adhesive rings to place electrodes on bare skin. The grip of the washer is better, when both, skin and holder, are degreased with alcohol prior to application.

Service

In case of any further questions please contact our local distributor or info@easycap.de. Please have the serial number of your cap at hand (syntax Axxxxx, located on the backside label of the cap).

¹ Electrode Nomenclature according to:

Oostenveld, R. & Praamstra, P. The five percent electrode system for high-resolution EEG and ERP measurements. *Clinical Neurophysiology* 2001; 112: 713-719