




64Ch Wet-Sponge R-Net for actiCHamp Plus

Including Cable-Set for Connection to actiCHamp Plus

Electrode Layout and Channel Assignment

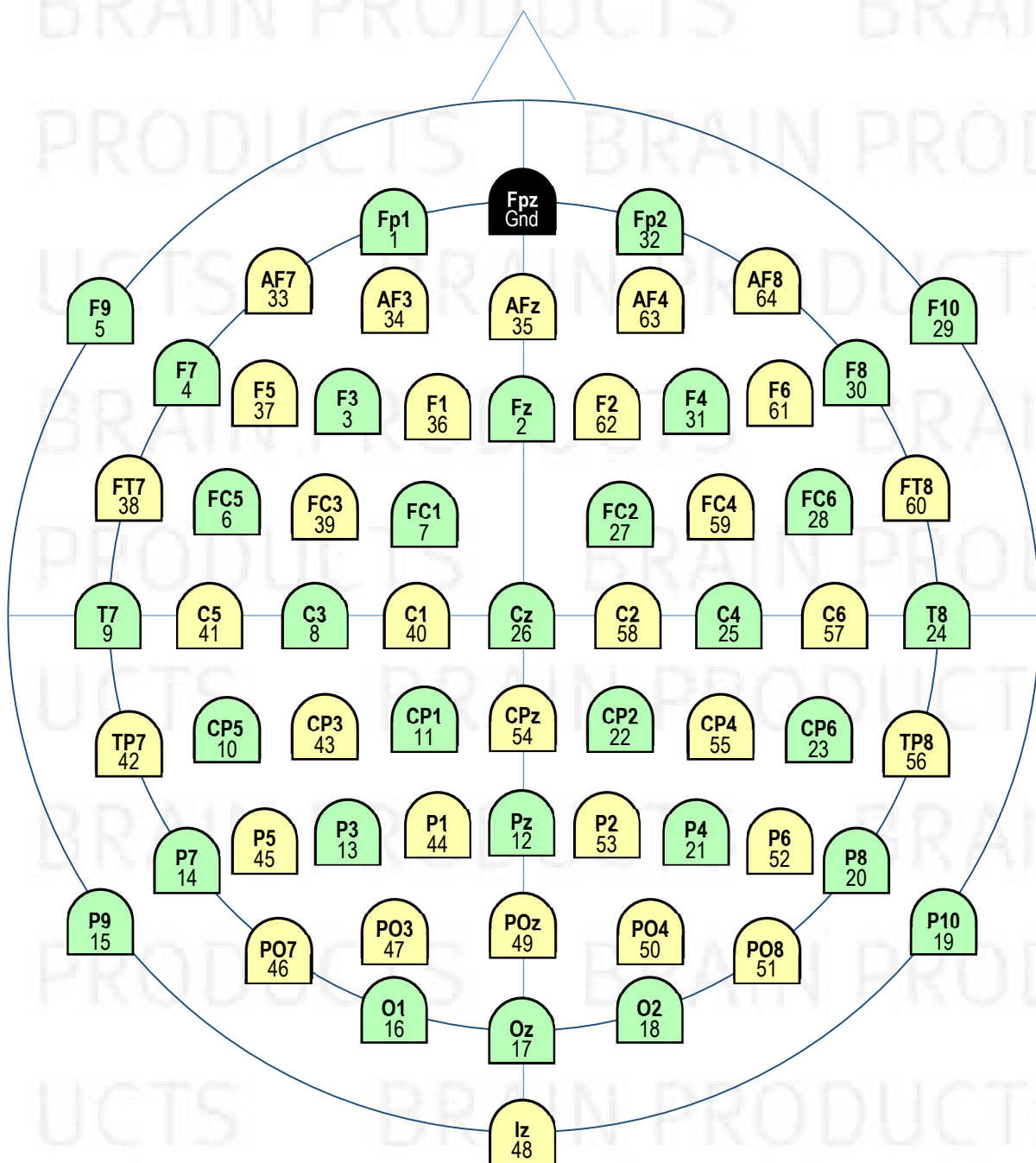

Details for Users

Ordering Information

For ordering please give **Article Number** and **Size** (e.g. RNPA-AC-64, 57):

- Article Number: **RNPA-AC-64**
- Size (given in cm head circumference):
 - 61** AXL, adult extra large
 - 59** AL, adult large
 - 57** AM, adult medium
 - 55** AS, adult small
 - 53** PL, pediatric large
 - 51** PM, pediatric medium
 - 49** PS, pediatric small
 - 47** IXXL, infant extra extra large
 - 45** IXL, infant extra large
 - 43** IL, infant large
 - 41** IM, infant medium
 - 39** IS, infant small
 - 37** NL, neonate large
 - 35** NM, neonate medium
 - 33** NS, neonate small

The catalogue-number comprises the cap as described including a cable-set to connect to actiCHamp Plus, serial number, this document; a memory-stick with manual and bvef-file, and the saltwater net accessory kit SNAK, consisting of 35 spare sponges and 5 pedestals, all packed in a labelled cardboard box. For further information about accessories or consumables, please visit our website or contact our local distributor.

Please adhere to Manual

The manual explains in detail how to prepare, mount, and adjust the cap. Maintenance, cleaning, and disinfection are covered, and what to do when repair is needed. There are several features to improve fit. Please adhere to manual.

Layout

The layout for this R-Net for actiCHamp Plus is equal to R-Nets for other amplifiers, but comes without a dedicated Ref-Electrode. Thus the electrode-site FCz stays empty.

Electrodes

The electrodes have Ag/AgCl sensors. They are number-labelled at the sensor end. The cables leave the net occipitally, pointing downwards. The length of cable tree is approx. 120 cm.

Termination and Cables to actiCHamp Plus

The cable tree is terminated into a connector box. From here the caps are connected to actiCHamp Plus with one flat-ribbon-cable per 32 channels plus one Gnd-cable. These cables are needed once per amplifier, not per net. For a first R-Net choose catalogue-number BP-130-5210 (Brain Products) / RNPA-AC-64 (Easycap) which comes **with** cables, for further nets choose BP-330-5210 / RNP-AC-64 which come **without** cables.

If further cables are needed they can be re-ordered from Brain Products or Easycap:

	Brain Products Cat-No.	Easycap Cat-No.
Gnd Cable	BP-245-1500	BP-245-1500 CABLE B4P
Flat-cable for Ch 1-32	BP-245-1100	E81-22
Flat-cable for Ch 33-64	BP-245-1110	E81-25

Table of ThetaPhi-Coordinates for RNPA-AC-64

Ch-No.	Name	Theta	Phi
1	Fp1	-90	-72
2	Fz	45	90
3	F3	-60	-51
4	F7	-90	-36
5	F9	-113	-36
6	FC5	-69	-21
7	FC1	-31	-46
8	C3	-45	0
9	T7	-90	0
10	CP5	-69	21
11	CP1	-31	46
12	Pz	45	-90
13	P3	-60	51
14	P7	-90	36
15	P9	-113	36
16	O1	-90	72
17	Oz	90	-90
18	O2	90	-72
19	P10	113	-36
20	P8	90	-36
21	P4	60	-51
22	CP2	31	-46
23	CP6	69	-21
24	T8	90	0
25	C4	45	0
26	Cz	0	0
27	FC2	31	46
28	FC6	69	21
29	F10	113	36
30	F8	90	36
31	F4	60	51
32	Fp2	90	72
33	AF7	-90	-54
34	AF3	-74	-68
35	AFz	67	90
36	F1	-49	-68
37	F5	-74	-41
38	FT7	-90	-18
39	FC3	-49	-29
40	C1	-23	0
41	C5	-68	0

42	TP7	-90	18
43	CP3	-49	29
44	P1	-49	68
45	P5	-74	41
46	PO7	-90	54
47	PO3	-74	68
48	Iz	112	-90
49	POz	67	-90
50	PO4	74	-68
51	PO8	90	-54
52	P6	74	-41
53	P2	49	-68
54	CPz	22	-90
55	CP4	49	-29
56	TP8	90	-18
57	C6	68	0
58	C2	23	0
59	FC4	49	29
60	FT8	90	18
61	F6	74	41
62	F2	49	68
63	AF4	74	68
64	AF8	90	54
Gnd	Fpz	90	90

These values are standardized to a Theta of 90° for the plane through Fpz, T7, T8, Oz.

The signs follow this convention:

